



## **MST Mannequin Adhesive: Usage & Instructions**

MST Mannequin Adhesive is a liquid-based adhesive used to apply MST prosthetics to training mannequins. Available in a twist-top tube, it can be sealed and saved for later use. Working time is approximately 30 minutes once exposed to air (a hairdryer or other heat source can be used to speed the cure). MST Mannequin Adhesive is easily removed using rubbing alcohol.

### **Usage:**

To use MST Mannequin Adhesive, twist open the cap and squeeze a small amount of adhesive onto a clean, smooth working surface (such as MST Mixing Palette). MST Mannequin Adhesive should be applied with a cotton swab or similar absorbent-tipped tool. Cap MST Mannequin Adhesive when not in use, as it begins to harden when exposed to air.

### **Wound Application:**

Clean the area that the wound is to be applied to with an alcohol wipe or other oil-free astringent. Using a cotton swab or tool of your choice, apply a THIN layer of adhesive that is about the size of the wound. Allow the adhesive to dry COMPLETELY. Once the adhesive is no longer white and is completely clear, it is fully dry. This process may be expedited using a hair dryer.

Using the same procedure spread a THIN layer of adhesive to the back of the wound. Again, allow the adhesive to COMPLETELY dry. Once both sides of the adhesive are dry, firmly place the wound on the adhesive area. If the edges of the wound fold under, use a cotton swab dipped in rubbing alcohol to unroll them. A SMALL amount of Mannequin Adhesive may be used to further secure any free edges. Once again, allow to fully dry.

Use baby powder or translucent powder to cover the tackiness of any exposed adhesive.

### **Removal:**

MST Mannequin Adhesive may be easily removed with rubbing alcohol. Thoroughly saturate the wound with rubbing alcohol. Using an alcohol soaked wipe, rub the surface to remove exterior MST Mannequin Adhesive. Use a cotton swab soaked in alcohol to remove the prosthetic from the mannequin.

### **Cleaning:**

Clean any surface exposed to MST Mannequin Adhesive with rubbing alcohol. To remove clumped adhesive from wounds or other surfaces, submerge the item in rubbing alcohol and allow to soak until adhesive has loosened.

### **Warnings:**

Do not use MST Mannequin Adhesive around the eyes or mouth. MST Mannequin Adhesive is not easily removed from fabric or porous surfaces such as carpet, upholstery, and other fabrics.

### **Related Products:**

- MST Mixing Palette
- MST Prosthetics