



MST Nuts, Bolts & Screws Shrapnel: Usage & Instructions

An overall favorite of customers and staff alike, this shrapnel is designed to mimic the type of destructive objects found in an Improvised Explosive Device (IED). Made from a soft, flexible plastic, it can be glued to any MST prosthetic using MST Silicone Adhesive or be scattered around for a realistic effect.

Usage:

MST Nuts, Bolts & Screws Shrapnel is compatible with all MST Prosthetics. To adhere to wounds or other prosthetics, use MST Silicone Adhesive to secure the Nuts, Bolts & Screws Shrapnel in place. MST Nuts, Bolts & Screws Shrapnel may also be used to scatter throughout the training scenario for a realistic effect.

Removal:

Using firm pressure, gently pull the MST Nuts, Bolts & Screws Shrapnel out of the prosthetic. The Silicone Adhesive will not permanently adhere the Nuts, Bolts & Screws Shrapnel to the wound.

Cleaning:

MST Nuts, Bolts & Screws Shrapnel may be rinsed in soapy warm water to remove residue, simulated blood, and other contaminants.

Warnings:

While MST Nuts, Bolts & Screws Shrapnel is made from a soft, flexible plastic, some edge of the shrapnel may still be sharp. Use caution when adhering MST Nuts, Bolts & Screws Shrapnel in areas that may cause discomfort to the user (inner leg, inner arm, neck, underarm, etc.).

Related Products:

- MST Silicone Adhesive
- MST Palette
- MST Plastic Knife
- MST Silicone Prosthetics