



MST Odor Neutralizers: Usage & Instructions

MST Odor Neutralizers were developed in order to eliminate the unpleasant smells of MST Odors after the training scenario has been completed. MST Odors are very strong and can be hard to remove completely. MST Odor Neutralizers make this an easy task and are available in a variety of pleasant smells.

Usage:

To use MST Odor Neutralizers, remove the screw-top lid of the odor neutralizer bottle and replace with the spray-top lid. Once the spray-top lid is in place, the odor can be sprayed on any surface, MST Prosthetic, prop, or fabric that smells unpleasant. MST Odor Neutralizers can also be sprayed throughout the room to remove lingering scents.

Cleaning:

If spills occur, absorb as much liquid as possible. To remove residual oils, dilute the area with rubbing alcohol and absorb. Repeat as necessary.

Warnings:

Do not use MST Odor Neutralizers around the eyes, mouth, or directly on human skin. When used on human skin, irritation may occur. MST Odor Neutralizers are also not easily removed from human skin. MST Odor Neutralizers may not wash out of certain fabrics.

Related Products:

- MST Odors