



MST Simulated Urine: Usage & Instructions

Made from a liquid that will not evaporate during training, MST Simulated Urine is designed to make training scenarios more realistic. MST Simulated Urine comes in three colors that simulate different medical conditions: Regular, Dark & Blood.

Usage:

MST Simulated Urine is a standalone product and can be used in any training scenario. Put MST Simulated Urine on floors or other surfaces to simulate the presence of urine. Allow trainees to identify medical conditions based on the color of MST Simulated Urine used. MST Simulated Urine may also be used on clothing to simulate urine stains. MST Simulated Urine is compatible with MST Urine Odor.

Cleaning:

MST Simulated Urine may be cleaned by absorbing excess liquid and wiping up any residue using rubbing alcohol.

Warnings:

MST Simulated Urine may stain light clothing & surfaces. Test in an inconspicuous area before use. Do not use MST Simulated Urine in the mouth or eyes. MST Urine may make surfaces slippery during & after use.

Related Products:

- MST Urine Odor